

Burning CDs / DVDs with ShowCase and Windows Integrated Burning

NOTE: There are troubleshooting tips at the end of this document.

Who should use this guide: This guide gives instructions for burning CDs or DVDs on computers that DO include the Windows® integrated “CD Writing Wizard” for CDs or the DVD wizard for DVDs. If your computer DOES NOT have integrated burning, please see the separate guide for Burning CDs/DVDs Using a ShowCase Staging Folder.

WINDOWS 7 Users – Your computer should have both integrated CD and DVD burning. If the integrated burning has been disabled, please use the guide for burning with a staging folder.

WINDOWS XP and Vista Users – You should have integrated CD burning but NOT integrated DVD burning. Use this guide for burning CDs.

What you will need to make DICOM CDs or DVDs with ShowCase

- 1- ShowCase Premier must be installed on a computer that has an internal or external CD/DVD recorder. (The CD Burn button is not available in ShowCase Basic.)
- 2- You must have a PC that supports burning CDs (DVDs on Win 7 and later) using the Windows integrated CD Writing Wizard. (When you insert a blank CD a wizard comes up to ask what to burn to the CD)


To Copy Studies to CD with Windows XP, Vista, Win 7 integrated burning



1- On the Study Lists Dialog select the study (or multiple studies) you wish to put on the CD and click the Burn CD/DVD button. The top half of the Burn CD or DVD dialog has the integrated CD burning options.



2- If you want to include the ShowCase Onboard Viewer on your CD, you can check the box that says "Include Viewer". Uncheck the box if you do not want a viewer included.

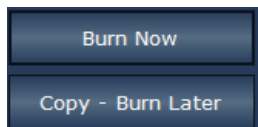

3- Click on "Burn Now" to burn the CD immediately. Or click on "Copy - Burn Later" to copy the study to the CD Burning folder without burning the CD yet. You can add more studies later. A “You have files waiting to be written to the CD” message may be displayed by Windows; this message can be ignored.



Folder contains 170.1 MB

4- If you did not burn the CD immediately, you can click on the "CD Burning" tab to see the studies waiting to be burned to CD. You can delete a study from the CD Burning folder at any time. You can copy additional studies to the folder. NOTE the size (in MB) of the studies in the staging folder. Be sure that this number is less than the capacity of your CD and delete studies if necessary.



5- If you did not burn the CD immediately, you can now click on the CD Burning tab and then on the Burn CD button on the Study Lists Window.

6- The CD Burn Wizard finishes producing the CD and then ejects the CD for you if you have configured it to do so.

CAUTION: Do not add studies to a CD that already has previously burned studies on the CD. Results are unpredictable. Make a new CD. Also, once you begin a burn, do not modify the contents of the CD burning folder outside of ShowCase.

Troubleshooting Tips

Note that only Windows 7 and later operating systems include integrated DVD burning. You will need to use the Staging Folder burn instructions (see separate guide) for operating systems earlier than Win 7.

If you do not have a Burn CD/DVD button on your Study Lists Window, then you are running either ShowCase Basic or ShowCase Onboard, software that does not include the ShowCase CD/DVD burning feature. You will need to upgrade to Premier if you want to use this feature.

If the “Burn Now” button is disabled, then your PC is not set up for integrated CD burning. This can often be fixed with a simple setting: Double-click on “My Computer” and then right-click on your CD drive and select “Properties”, then click on the “Recording” tab. Make sure this box is checked:

Enable CD recording on this drive

If the “Enable CD recording” option is not available then your PC may have software installed that disabled this capability. You will need to use the “staging folder” process described in a separate User Guide.

If your “Include Viewer” button is disabled, ShowCase did not find the Onboard Viewer files and you may need to re-install your software.