

Burning CDs from ShowCase Premier (Making CDs for referring physicians or for submitting for accreditation)

NOTE: There are troubleshooting tips that address CD burning problems at the end of this document.

What you will need to make DICOM CDs with ShowCase

- 1- ShowCase **Premier** must be installed on a computer that has an internal or external CD burner.
- 2- Your operating system must be Windows® XP or Vista. If you have Windows XP it must have the integrated CD Writing Wizard enabled. (The wizard is enabled if you can insert a blank CD and then drag files to the window representing the CD and click “write these files to CD”.)

ShowCase Premier can interact directly with the Windows XP or Vista integrated CD Writing Wizard. If your machine does not have integrated CD writing, or if you want to burn **DVDs**, please see the separate guide for *Burning DVDs from ShowCase Premier*.


Steps to Burn CDs (Windows XP or Windows Vista Integrated CD Writing)



1- On the Study Lists Dialog select the study (or multiple studies) you wish to put on the CD and click the **Burn CD/DVD** button. The top half of the Burn CD or DVD dialog has the integrated CD burning options.



2- If you want to include the ShowCase Onboard Viewer on your CD, you can check the **Include Viewer** box. Uncheck the box if you do not want a viewer included. You will want to include the viewer on accreditation CDs.

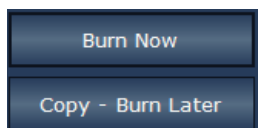

3- Click on **Burn Now** to burn the CD immediately. Or click on **Copy - Burn Later** to copy the study to the CD Burning folder without burning the CD yet so that you can continue to add or delete studies. A message that says “You have files waiting to be written to the CD” may appear; this message can be ignored.



Folder contains 170.1 MB

4- If you did not burn the CD immediately, you can click on the **CD Burning** tab in the Study Lists Dialog to see the studies waiting to be written to CD. You can delete a study from the CD Burning folder at any time. You can copy additional studies to the folder. NOTE the size (in MB) of the studies in the staging folder. Be sure that this number is less than the capacity of your CD and delete studies if necessary.



5- When you are ready to burn the CD, click on the **Burn CD** button while the **CD Burning** tab is forward.

6- The Windows CD Writing Wizard asks you to name the CD, then writes the CD and ejects it. ShowCase empties the burn folder for you at the end.

CAUTION: Do not add studies to a CD that already has previously burned studies on the CD. Results are unpredictable. Always make a new CD. Also, once you begin a burn operation, do not modify the contents of the CD Burning folder outside of ShowCase.

Troubleshooting Tips

If you do not have a Burn CD/DVD button on your Study Lists Window, then you are running ShowCase Basic or ShowCase Onboard software that does not include the CD/DVD burning feature. Go to www.TriITech.com or email an order to Sales@TriITech.com if you wish to purchase ShowCase Premier.

If the **Include Viewer** button is disabled, ShowCase did not find the Onboard Viewer files on your computer and you may need to re-install your software.

If the **Burn Now** button is disabled, then your PC is not set up for integrated CD burning. This can often be fixed with a simple setting: Double-click on “My Computer” and then right-click on your CD drive and select “Properties”, then click on the “Recording” tab. Make sure this box is checked:

Enable CD recording on this drive

If you insert the CD into a PC and the viewer does not launch automatically, the PC may not be configured to autorun applications from CDs. In this case, browse to the CD and double-click on file called Show_Studies. (This information is included in the README file on the DVD.)