



Upgrading to ShowCase 6.6

Overview

SHOWCASE® 6.6 contains a few modifications to support the new study archival behavior in the Image Center. If you are not using ShowCase with Image Center it is unlikely you will notice any difference.

Doing The Upgrade

The upgrade is straightforward, but it does require that you have Administrator privilege to do it.

1. Download the new ShowCase DICOM Viewer installer from here: <https://www.triltech.com/download.html>
2. Uninstall your existing ShowCase 6.x software. All of your data, configuration, and license settings will still be on the computer.
3. Run the installer you downloaded in step 1.
4. Start ShowCase

SHOWCASE® is a registered trademark of Trillium Technology, Inc.